



SIZE & SHADE GUIDE

The Good Fit® Denture Tray is currently available in three sizes (with an upper and lower tray of each size). The tooth forms on the trays function primarily as a substitute for wax occlusal rims, but they can also be used as an in-the-mouth reference for helping you and your patient select tooth molds for the final prosthesis (for example, by assessing whether the tooth forms on the selected tray appear too big or too small in the patient's mouth - or whether the shade is too dark or too light - and then using the information below to help choose the final denture teeth).

Size:	Base Dimensions*		Tooth Dimensions:		
	Length	Width	Six Anterior Width	Central Width	Central Length
Upper Trays					
Small	50mm	44mm	44mm	7.5mm	8.4mm
Medium	56mm	47mm	48mm	8.1mm	10mm
Large	60mm	48mm	50mm	8.6mm	11mm
Lower Trays					
Small	48mm	46mm	33mm	4.2mm	9.2mm
Medium	53mm	48mm	36mm	5.0mm	9.9mm
Large	58mm	50mm	38mm	5.6mm	10.4mm

* **Length:** anterior to posterior. **Width:** center of ridge in posterior region of tray

TOOTH SHADE

The tooth shade (and shade of the tray itself) on all tray sizes is approximately A2 on the Vita® Classical Shade Guide.

617-973-5136

www.goodfit.com